



Easy Swiveling for Car or Bed

Travel in comfort and safety



The TwistAssist® makes it easy to enter or leave a car as a passenger or a driver. Its soft cushions remain comfortable to sit upon for even the longest of journeys. The non-slip base keeps the cushions in place; they will mould to the shape of a bucket seat and still swivel easily.



When entering the car sit fully over the cushion. You will then swivel with your back to the rear of the seat. If you can only sit partially on the TwistAssist® cushion, wriggle into the back. If this is difficult you will be assisted by the use of a One Way Slide Lite from SlideAssist®. Place the One Way Slide over the TwistAssist® approximately in line where you will sit. This will help you slide into the correct position either before or after you swivel. It will grip in place and prevent sliding forward.



A person entering or leaving a car may also be assisted by using a Handibar® inserted into the U-bolt on the car door. Gripping the handle provides extra support during the transfer.



To swivel onto a bed, sit on the TwistAssist®, raising legs swivel onto the bed. Lie flat. Remove the TwistAssist®, with a log roll. Reverse procedure to swivel off.

Washable at 40°C or brush lightly or wipe with a damp cloth. Do not use chlorine or fabric softeners.

Available from